



TERGAL 800® Detergent Instructions for Use P/N: 13020

Read the complete Instructions for Use and the Safety Data Sheet (SDS) prior to use. It is also recommended to read the instrument manufacturer's reprocessing instructions and, if applicable, the automated endoscope washer / disinfectant instructions for use.

Warning:

- Causes serious eye irritation.

Cautions:

- Wash hands thoroughly after handling.
- Wear eye protection / face protection.
- IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. If eye irritation persists, get medical advice / attention.
- Tergal 800 is a detergent; it is NOT a High-Level Disinfectant.



Sample Image
(Subject to Change)

Description

Tergal 800 Detergent is a low foaming, versatile, free rinsing detergent.

Note: Tergal 800 Detergent does not contain any enzymes and require specifications for time and temperature to be effective.

Directions for Manual Use

Tergal 800 Detergent is intended for reprocessing of surgical instruments, immersible endoscopes, and related accessories. For manual reprocessing, mix a solution of 0.7 fluid ounces of Tergal 800 Detergent (concentrate) per gallon of fresh water.

Directions for use in the Custom Ultrasonics Sinks

Custom Ultrasonics Scope-Assist Flushing Sinks and Pre-Processing Sinks are equipped with an automated dispensing timer that dispenses the Tergal 800 Detergent. Refer to the appropriate sink Operator's Manual for complete directions for use.

Directions for use in the Custom Ultrasonics System 83 Plus™ Washer / Disinfectant

These systems are equipped with automated detergent dispensing that dispenses the appropriate concentration of Tergal 800 Detergent. Refer to appropriate System 83 Plus Operator's Manual for complete directions for use.

Note: If the Tergal 800 Detergent bottle has ≤ 1 " of detergent remaining, replace with a new bottle of detergent so that a correctly metered dose can be dispensed.

Packaging

4 Bottles per Case (4 Gallons)
1 Gallon per Bottle

Storage and Disposal

Tergal 800 Detergent should be stored between 35°F and 100°F in the closed container. Discard unused detergent according to local regulations.

Expiration

Eighteen (18) months from date code stamped on label.